# **Home Care Technical Specification**

## **Fold Down Shower Seat**

## **DN7110**

### **Stock Numbers:**

- DN7110 Fold down Shower Seat
- DN7110OWB Fold Down Shower Seat, Old World Bronze

### **Materials:**

#### Fold Down Shower Seat is made of Aluminum and Teak Wood

CARE:

Clean the Teak seat with a warm, damp cloth. The natural properties of Teak allow for the wood to weather over time in the environment. If you desire to keep the current look of the wood, treat the teak with a certified Teak oil.

ASSEMBLY TESTED TO CONFIRM ADA COMPLIANCE



#### WARNING:


201593 500004625

5/06/0022 kelessed 01 TO PREVENT SERIOUS OR FATAL INJURY: Insure product is assembles and used properly. FOR USE ONLY BY PERSONS WEIGHING LESS THAN 400 POUNDS. Failure to follow correct assembly procedures can cause the seat to become unstable. DO NOT jump on, stand on or tilt. Contact Moen immediately at 1-800-882-0116 if any components are missing or damaged. Do not modify or substitute any components.

FOR QUESTIONS OR ASSISTANCE WITH ASSEMBLY CONTACT Moen AT 1-800-882-0116. Consult healthcare professional for advice and instructions specific to user's needs before use.



## Dimensions:

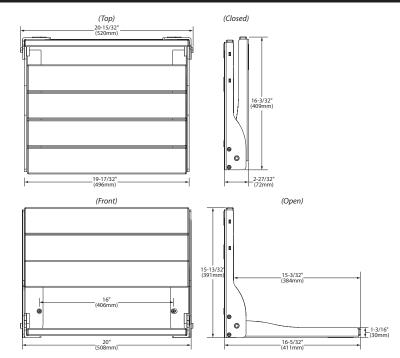

#### **Installation Instructions:**

- 1. Remove the top two back rest panels to reveal the top four mounting holes.
- 2. Identify the mounting location of the shower seat making sure the shower seat is level.
- 3. Mark the six mounting hold locations once the desired location is determined.
- 4. Set aside the shower seat and drill 1/8" pilot hole through tile in each mounting location. Drill a 1/4" hole through the tile only at the same locations to allow the wood screw to pass through.
- 5. When complete, ensure the wall is clear of any deprise.
- 6. Secure the shower seat to the wall, using the provided six wood mounting screws. Start with the top two mounting hole locations.
- 7. Once all screws have been installed and tightened, slide the back panels into place.
- 8. Installation is complete Test the seat to make sure it is secure.

#### Stock Number:\_\_\_\_\_ By Moen Incorporated

Shower Seat

25300 AL MOEN DRIVE • NORTH OLMSTED, OH 44070 CORPORATE HEADQUARTERS: (800) 321-8809 • FAX: (800) 848-6636 INTERNATIONAL: (440 ) 962-2000 • FAX: (440) 962-2726 CUSTOMER SERVICE (USA): 800-882-0116 • FAX: (888) 379-2720 WWW.MOEN.COM ©Copyright Moen Incorporated